

# SPRING L.E.D. HEADLAMP



Adjustment Knob  
Push in and turn



On/Off Switch and Plug

Remove dock station with screw driver, then add belt clip to dock station using screws.

Step 1



Step 2



Installing Extension  
Cord/Belt Clip

Step 3

Lastly, plug male end of extension cord into dock station. Attach dock station to belt using clip and unit is ready to use.

Install clamp onto extension cord, plug female end of extension cord into male end on headset. Then bolt clamp onto headset as shown on the right.

## Battery Removal



1. Push battery to the right and lift up to remove. (Top Picture)
2. To install, lay battery in grooves and push to the left till it clicks in place. (Bottom Picture)

### **Instructions: Mount Digital Battery**

Align the metallic contacts of the battery to the inserting direction and insert it against electrode gate protection switch.

Connect the power plug of the battery-mounted charger to the power socket or car cigarette.

If CHARGE indicator orange or red lights, charging begins.

If CHARGE indicator green lights, charging is completed normal charge.

If charger don't connect the power, CHARGE indicator green lights.

If you extend charging time about 30-60 minutes after normal charge is completed, you can get longer service time.

### **Warning:**

- Charger shall be unplugged from the power socket if it is not used for an extended period of time.
- It is normal if the product gets slightly hot in suing.
- Don't expose the product to rain or humidity to prevent from electric shock
- Keep the product out of reach of children.